H2RESQ

Your rescue partner.

SKED[®]

THE COMPLETE RESCUE SYSTEM

⚠ WARNING

Read and understand operator's manual and all other safety instructions before using this equipment.

MANUAL
HANDLEIDING
HANDBUCH

SKED® SAVES LIVES

Congratulations on your purchase of the Sked® Rescue System. You will find it a valuable tool that will enable rescues to be performed safer, faster, and easier. The Sked® Stretcher is an entirely different concept in stretcher design. Therefore, potential rescuers should practice and familiarize themselves with the Sked® prior to using it on an actual rescue.

The Sked® Stretcher provides excellent patient support and protection. However, it is NOT designed as a spinal immobilization device. If a spinal injury is suspected, secure the patient to an approved spinal immobilization device prior to placing him/her in the Sked®. The Sked® Stretcher will accommodate long and short backboards, scoop stretchers, Oregon Spine Splint II™ and most other immobilization equipment. A backboard must be used in conjunction with the Sked® Stretcher on patients who have sustained injuries to the shoulder area.

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TO UNROLL SKED® STRETCHER AND LAY FLAT

*STEEL BUCKLES

*COBRA BUCKLES

1. REMOVE SKED® FROM PACK AND PLACE ON GROUND.

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2. UNFASTEN RETAINER STRAP. STEP ON FOOT END OF SKED® AND UNROLL COMPLETELY TO OPPOSITE END.

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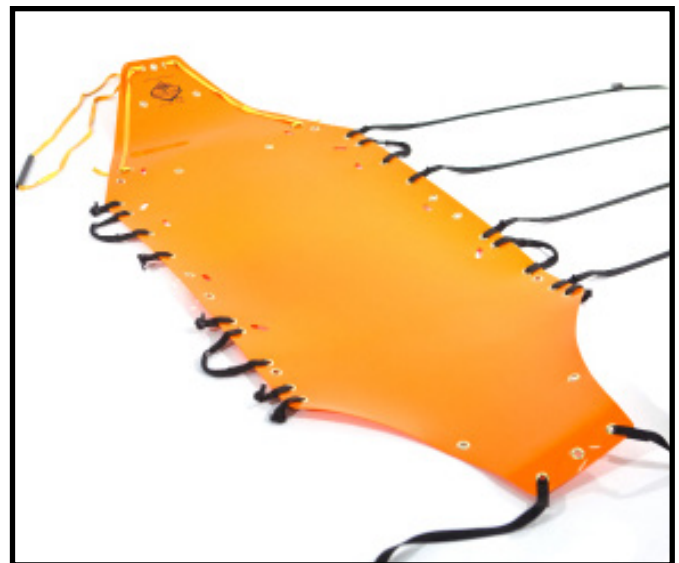
3. BEND THE SKED® OVER YOUR FOREARM AND BACK ROLL. REPEAT WITH OPPOSITE END OF STRETCHER. IN COLD WEATHER, IT MAY BE NECESSARY TO REVERSE ROLL IT TWICE.

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PLACE A PATIENT ON THE SKED®: PULL METHOD (FEET FIRST)

1. PRIOR TO DRAGGING A PATIENT ONTO THE SKED® "FEET FIRST". CURL THE HEAD END DOWNWARD TO FORM A RAMP AND TO PREVENT SNAGGING PATIENT'S CLOTHING.

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2. TUCK THE DRAG WEBBING UNDER THE SKED®. GRASP PATIENT'S LEGS AND DRAG PATIENT WHILE KEEPING LEGS LOW.

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3. CONTINUE DRAGGING UNTIL HIS UPPER CHEST IS EVEN WITH THE UPPER CROSS STRAP.

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4. SECURE ALL BUCKLES AND ADJUST ALL STRAPS IN PREPARATION FOR TRANSPORT.

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PLACE A PATIENT ON THE SKED®: PULL METHOD (HEAD FIRST)

1. CURL FOOT END OF SKED® DOWNWARD TO FORM A RAMP AND PULL FOOT END STRAPS TO THE SIDE.

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2. USING YOUR FOREARMS TO PREVENT LATERAL MOVEMENT OF PATIENT'S HEAD, GRASP THE UNDERARMS AND DRAG IN-LINE WITH THE SPINE ONTO THE SKED®.

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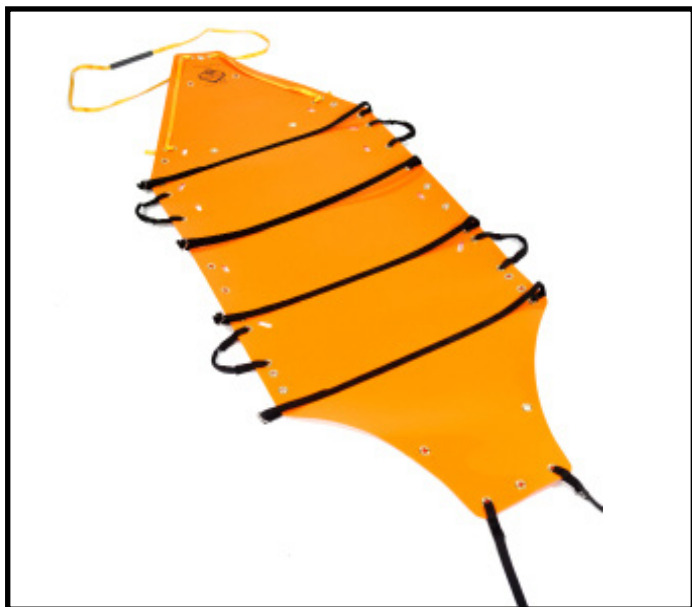
FASTEN THE STRAPS: COBRA QUICK RELEASE BUCKLES

1. PRIOR TO LOADING THE SKED® RELEASE ALL COBRA BUCKLES AND PULL STRAPS TO ONE SIDE.

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2. LIFT SIDES OF SKED® AND FASTEN THE FOUR CROSS STRAPS TO BUCKLES DIRECTLY OPPOSITE TO THE STRAPS AND PULL THEM TIGHT.

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3. FASTEN FOOT END COBRA BUCKLES AND ADJUST STRAPS.

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4. DO NOT CRISS-CROSS STRAPS ON THE SKED®. IT WILL CAUSE THE SKED® TO BEND IN THE MIDDLE.

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FASTEN THE STRAPS: STEEL BUCKLES

1. PRIOR TO LOADING THE SKED® RELEASE ALL STEEL BUCKLES AND PULL STRAPS TO ONE SIDE. TO RELEASE, LIFT THE LEADING END OF THE BUCKLE TO LOOSEN WEBBING. PULL THE WEBBING OUT OF THE BUCKLE.

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2. FASTEN THE 4 CROSS STRAPS TO BUCKLES DIRECTLY OPPOSITE TO THE STRAPS. TO CONNECT, SEE STEP #3 TO LACE BUCKLES, THEN TIGHTEN STRAPS.

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4. FEED THE STRAPS THROUGH UNUSED GROMMETS AT FOOT END OF SKED AND FASTEN TO BUCKLES THEN ADJUST STRAPS.

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HORIZONTAL LIFT/DESCENT

1. INSERT ONE END OF HEAD STRAP THROUGH ROUND SLOT AT HEAD END OF SKED®.

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2. BRING STRAP UNDER SKED® AND THROUGH ROUNDSLOT ON OPPOSITE SIDE OF SKED®.

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3. EQUALIZE STRAP. REPEAT PROCEDURE WITH OTHER STRAP AT FOOT END OF SKED®.

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4. EQUALIZE ALL FOUR STRAPS AND SECURE ENDS WITH LARGE STEEL LOCKING CARABINER.

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VERTICAL LIFT/DESCENT

1. PRIOR TO HOISTING. THE HEAD END MUST BE CURLED OVER THE PATIENT'S HEAD AND THE DRAG HANDLE WEBBING TIED TO THE SECOND CROSS STRAP TO PROTECT PATIENT'S HEAD FROM FALLING OBJECTS.

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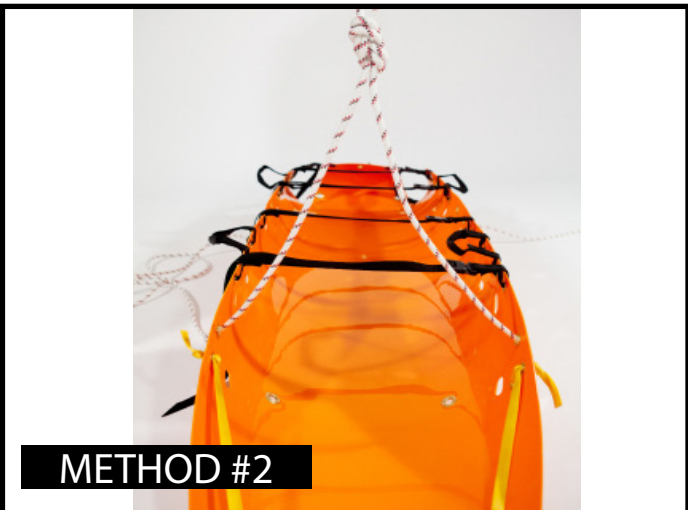
METHOD #1

2. INSERT VERTICAL LIFT SLING ROPE THROUGH THE GROMMETS ABOVE PATIENT'S HEAD.

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METHOD #2

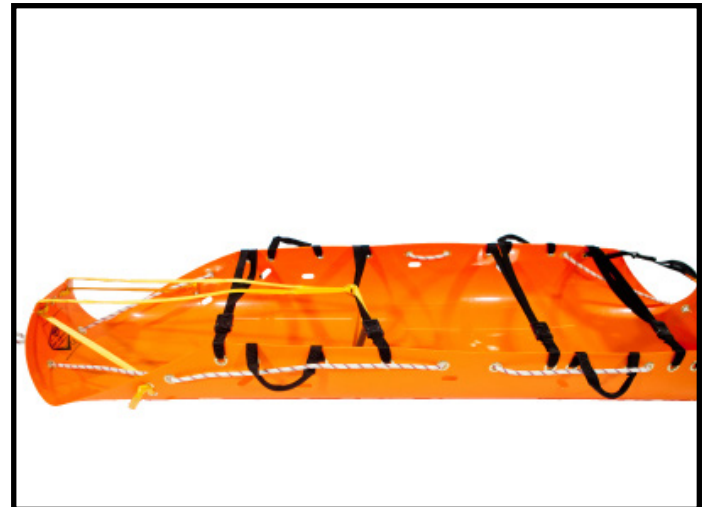
3. FOR EXTREMELY SMALL OPENINGS, LEAVE THE HEAD END EXTENDED. START THREADING THE ROPE THROUGH THE GROMMETS BY THE PATIENT'S SHOULDERS.

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4. CONTINUE THREADING THE ROPE ON EACH SIDE OF THE SKED® AS SHOWN IN PICTURE ABOVE.

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VERTICAL LIFT/DESCENT

5. THREAD THE ROPE THROUGH THE BOTTOM GROMMETS FROM THE INSIDE-OUT AS SHOWN ABOVE.

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6. TIE THE ROPE IN A SQUARE KNOT AT THE LOWER END OF THE SKED®.

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7. PASS THE ENDS OF THE ROPE THROUGH THE LOWER CARRY HANDLES FROM THE OUTSIDE-IN AND TIE ANOTHER SQUARE KNOT.

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8. TIE AN OVERHAND KNOT ON EACH SIDE OF THE SQUARE KNOT FOR SAFETY.

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BACKPACK/TOWING HARNESS

1. ATTACH ONE END OF THE TOW STRAP TO THE BACKPACK. BACKPACK IS USED AS TOWING HARNESS.

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2. ATTACH THE OTHER END OF THE TOW STRAP TO SKED® DRAG WEBBING. THIS ALLOWS "HANDS FREE" DRAGGING OF THE SKED®.

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ACCESSORIES: TOW STRAPS/REMOVABLE WEBBING HANDLES

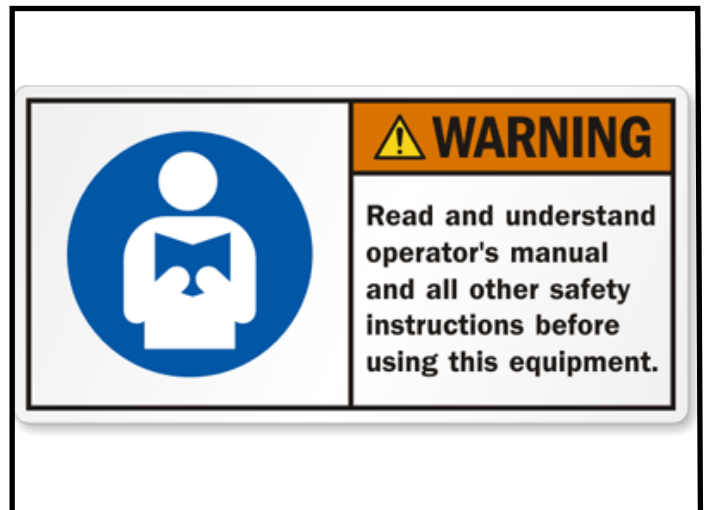
3. TOW STRAP IS ATTACHED TO CARRY HANDLES ENABLING 3 OR 4 RESCUERS TO DRAG THE SKED®.

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4. ERINNERUNG

ROLLING UP THE SKED® STRETCHER

1. LAY STRETCHER OUT FLAT WITH ALL CROSS STRAPS CONNECTED. **STEEL DOUBLE BAR BUCKLES: PLACE RETAINER STRAP WITH INSIDE OF BUCKLE FACING UP UNDER FOOT END OF SKED®. (SEE PHOTO IN STEP 4)

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3. WHILE HOLDING YOUR KNEE ON THE ROLLED SKED®, FASTEN THE COBRA BUCKLES OF THE FOOT END STRAPS TO HOLD THE SKED® IN A TIGHT ROLL.

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2. STARTING AT THE HEAD END, ROLL THE SKED® UP AS TIGHT AS POSSIBLE. CONTINUE TO ROLL THE SKED® USING KNEE TO KEEP THE ROLLED STRETCHER SMALL.

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4. WHEN USING THE SKED® WITH STEEL DOUBLE BAR BUCKLES, FASTEN THE PRE-PLACED RETAINER STRAP TO BUCKLE AND PLACE SKED® STRETCHER IN BACKPACK.

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NOTES

WARNING: SKED STRETCHERS CANNOT BE USED AS A BODY SUPPORT DEVICE IN A FALL PROTECTION SYSTEM.

SKED® FAQs

What does “SKED” stand for?

 Though we spell it with capital letters, SKED® is not an acronym. It came from fusing two words: “Sled” and “Skid.” The early idea behind the product was that it was a SLED that SKIDDED across all types of terrain. Initially, the SKED® was used as a game carrier to tote wild game back to a hunter’s camp. It has since morphed into the life-saving device that’s used today, worldwide. Like many iconic products that are the first of its kind in the market, the brand has come to epitomize the product itself.

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When does the SKED stretcher need to be replaced?

 We have a max time limit off 10 years, when the SKED® needs to be replaced because different rescue professionals use it in different ways for different lengths of time. If there is any damage to your stretcher, please reevaluate if you should be using it. If you have questions do not hesitate to call us.

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What should I do if my SKED® is missing a grommet?

 SKEDCO uses two grommet sizes that can be found at most local hardware stores. There are two different sizes: the smaller grommet is a size #3, and the larger is a size #5.

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What should I do if the straps on my SKED® are starting to fray?

 We have replaceable straps – the SK-208C. The SKED-EVAC® Aluminum Side Release Buckle Conversion Kit features Austrian-made Skedco/Cobra side release buckles. These buckles are dependable and strong enough to meet the needs of securing a patient into a SKED® stretcher without the danger of breaking the buckle or accidental release.

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SKED® FACT SHEET

THE SK-200-OR SKED® BASIC RESCUE SYSTEM – INTERNATIONAL ORANGE COMES WITH:

Available with Flotation and the Oregon Spine Splint II (OSS) options

Although the SKED system works very well for virtually all rescue situations, it excels in confined space, urban, military and industrial rescue.

MAX SAFETY WORK LOAD: SWL 250KG

SKED® FACT SHEET

SIZE

Rolled in cordura pack – 23 cm diameter x 91cm length

Laid out flat – 91cm x 244cm

WEIGHT 9kg. complete with accessories listed below.

MATERIAL Low density E-Z glide polyethylene plastic.

** Nylon webbing

** Solid brass grommets

** Steel buckles

TEMPERATURE RANGE Begins to melt 230 degrees C to 260 degrees C

** Usability to - 84 degrees C without becoming brittle.

HORIZONTAL LIFT SLINGS Webbing tensile strength 45 KN.

VERTICAL LIFT SLING Rope tensile strength 31KN.

SYSTEM COMPONENTS SKED STRETCHER - BASIC RESCUE SYSTEM INCLUDES:

** SKED Stretcher

** Cordura Back Pack/Towing Harness

** Horizontal Lift Slings

** Vertical Lift Sling (PMI E-Z Bend 9mm rope)

** Large steel locking carabiner (SMC)

** Tow Strap

Available with Flotation and the Oregon Spine Splint II (OSS) options

Cleaning - Disinfection

Drying.

Storage

Do not store in:

- Wet environment
- Humid environment
- Vibrations
- UV Radiation
- Sharp Edge Situation

Keep product cool & dry

SKED® MAINTANANCE

INSPECTION FORM VISUAL CHECKING OF THE SKED STRETCHER

TEXTILE PART : Condition of webbing (cuts, wear, burns) Condition of stitching (broken, pulled, or worn threads)
 METALLIC PART : Condition of anchor-rings (deformation, marks, cracks, wear, corrosion) Condition of the adjustment buckles deformation, marks, cracks, wear, corrosion)
 PLASTIC PART: : Plastic sheet (deformation, marks, cracks, wear, corrosion)

OPERATIONAL CHECK G TM TR R C

G: Good / TM: To Monitor / TR: To Repair / R: Reject / C: Comment

Date of inspection :Date of next inspection :

Inspected by.....Company name:.....

Historical Check:.....

Year of manufacturing:.....

Date off Purchase:.....

Ind. serial nr:.....

PRODUCT	G	TM	TR	R	C
BODY STRETCHER					
HORIZONTAL SLINGS					
VERTICAL ROPE					
KARABINER					
EXTRA HANDLES					
TOW STRAP					
GROMMETS					

COMMENTS.....

VERDICT: The product is fitto remain in service. The product is unfitto remain in service

In addition to routine checks for each use, Stretchers should regularly undergo a detailed inspection by a competent person. Skedco Europe recommends an inspection every 12 months and after any exceptional event in the life of the product.

Inspections that do not come into any of the categories listed above, any of which would require the systematic rejection or the component, namely:

- Component has received forces resulting from a fall of factor 1 or more.
- Component has had more than 6 months of intensive use, 12 months of normal use, 10 years of occasional use.

The inspector accepts no responsibility in the case of omission or inaccuracy in the information concerning the checking of the component's history, which must be done by the client.

SKED® INDIVIDUALLY NUMBERED/CERTIFIED

SCAN FOR
INSTRUCTION

MODEL _____
COLOUR **OR** _____
SERIAL NR **RESQ** _____
Your rescue partner.

COLOUR

INDIVIDUAL SERIAL NUMBER

Stretcher meets the requirements
of DIN EN 1496:2007

www.skedco.eu

For Example the Serial Number is 2102012

21= year of manufacturing / 02=week off purchase / 012=unique number

Colour: OR=International Orange / CB=Coyote Brown / GR=Green

By scanning the QR Code with a free app on your smartphone, you can instantly download the manual on your phone or tablet.

TRANSLATION

(1) **Type Examination Certificate**

(2) No. of the Type Examination Certificate: **ZP/B111/21**

(3) Product: **Rescue System**
Type: **Sked Rescue System**

(4) Manufacturer: **H2ResQ BV**

(5) Address: **Industrieweg 36-04, 3606 AS Maarsse, The Netherlands**

(6) The design of this product and any acceptable variation thereto are specified in the schedule to this Type Examination Certificate.

(7) The certification body of DEKRA Testing and Certification GmbH certifies that this product complies with the fundamental requirements of the standard listed under item 8 below. The examination and test results are set out in the report PB 21-098.

(8) The requirements of the standard are assured by compliance with
DIN EN 1497:2007

(9) This Type Examination Certificate relates only to the design, examination and tests of the specified product in accordance to the standard list. Further requirements of the Directive apply to the manufacturing process and supply of this personal protective equipment. These are not covered by this certificate.

(10) This Type Test Certificate is valid until 2026-07-01.

DEKRA Testing and Certification GmbH
Bochum, 2021-07-02

signed: Kilisch
Managing director

We confirm the correctness of the translation from the German original.
In the case of arbitration only the German wording shall be valid and binding.

Managing director

Page 1 of 2 of ZP/B111/21 - 342332000

This certificate may only be published in its entirety and without any change.
DEKRA Testing and Certification GmbH, Handwerksstr. 16, 70565 Stuttgart, Germany
Certification Body, Dinnendahlstraße 9, 44809 Bochum, Germany
Phone +49 234 3696-400, Fax +49 234 3696-401, DTC-Certification-Body@dekra.com

H2RESQ

Your rescue partner.

SKED[®]

THE COMPLETE RESCUE SYSTEM

H2-ResQ
Industrieweg 36 - 04
3606 AS Maarssen
The Netherlands

T+ 31 346 - 555 760

info@h2-resq.com
www.skedco.eu

WARNING

Read and understand operator's manual and all other safety instructions before using this equipment.